

In the presence of 5% blood serum and a contact time of 5 minutes, Contec Citric Acid Disinfectant is effective against:
GERMICIDAL: Enterococcus faecalis Vancomycin Resistant, Escherichia coli O157:H7, Listeria monocytogenes, Pseudomonas aeruginosa, Salmonella enterica, Staphylococcus aureus, Staphylococcus aureus – MRSA, Staphylococcus epidermidis - MRSE
VIROCIDAL*: Adenovirus Type 2, Feline Calicivirus [surrogate for Norovirus], Herpes Simplex Type 1 Virus, Herpes Simplex Type 2 Virus, Human Immunodeficiency Virus Type 1, Influenza A Virus [Strain Hong Kong], Pandemic 2009 H1N1 Influenza A Virus [formerly called swine flu], Respiratory Syncytial Virus, Rotavirus, Vaccinia Virus
FUNGICIDAL: Trichophyton mentagrophytes
In the presence of 5% blood serum and a contact time of 10 minutes, is effective against:
GERMICIDAL: Acinetobacter baumannii
VIROCIDAL: Canine Parvovirus [CPV], Hepatitis B Virus [HBV], Poliovirus Type 1
TUBERCULOCIDAL: Effective against Mycobacterium bovis [BCG] in 5 minutes at 20°C/68°F on previously cleaned hard non-porous inanimate surfaces.

SANITIZER: Staphylococcus aureus and Klebsiella pneumonia in 1 minute.
DIRECTIONS FOR USE: It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. Product is ready to use. **DO NOT DILUTE.** Pre-clean areas with gross filth and heavy soil prior to disinfection.
TO DEODORIZE: Apply to surfaces as needed. **TO CLEAN:** Apply Contec Citric Acid Disinfectant undiluted to hard, non-porous surfaces by spraying, wiping with a cloth or towel until clean. Spot check to test for compatibility. **TO SANITIZE:** Wet the surface. Let stand for one minute. Wipe or let air dry. No rinsing or wiping is required. **TO DISINFECT:** Apply Contec Citric Acid Disinfectant undiluted to hard, non-porous surfaces with a sprayer, cloth, mop or brush, or by immersing equipment [item] to be disinfected.] Wet all surfaces thoroughly. The surface must remain wet with Contec Citric Acid Disinfectant for at least 5 minutes. **Contact time in accordance with organisms listed on the distributed product label.** Wipe or let air dry. No rinsing or wiping is required, except on direct food contact surfaces which require a potable water rinse after treatment. **Toys:** Use only on hard non-porous surfaces. Rinse toys with potable water.

FUNGICIDAL DIRECTION: This product is effective against Trichophyton mentagrophytes [athlete's foot fungus and a cause of ringworm] on inanimate surfaces in locker rooms, dressing rooms, exercise facilities, and shower and bath areas. Follow disinfection directions.

***CONTEC CITRIC ACID DISINFECTANT KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in all settings in which there is an expected likelihood of soiling of hard, non-porous surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 HIV-1 associated with AIDS.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATING SURFACES/OBJECTS SOILED WITH BLOOD AND BODY FLUIDS WHICH MAY CONTAIN HIV-1 [AIDS virus]. PERSONAL PROTECTION: When handling objects soiled with blood or body fluids use disposable latex gloves, gowns, mask and eye coverings.

CLEANING PROCEDURE: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant. **CONTACT TIME: Do Not Dilute.** Leave pre-cleaned surfaces wet for 5 minutes using mop, cloth or sponge to mitigate HIV-1 virus. Use a 10 minute contact time to kill all bacteria, viruses* and fungi listed on the label. **Contact time in accordance with organisms listed on the distributed product label.**

DISPOSAL OF INFECTIOUS MATERIAL: Blood and other fluids should be autoclaved and disposed of according to Federal, State and Local regulations for infectious waste disposal.

SDS # CAD1284

Available exclusively from:
Contec, Inc. / Sporidicin Division
 525 Locust Grove
 Spartanburg, SC SC 29303 USA
 1-864-503-8333
 www.contecinc.com

2609-01/17



DECONTAMINATION OF CRITICAL OR SEMI-CRITICAL DEVICES PRIOR TO TERMINAL STERILIZATION/HIGH LEVEL DISINFECTION: Contec Citric Acid Disinfectant is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that [1] is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or, [2] contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

AREAS OF USE: Athletic facilities, classrooms, daycare centers, dialysis units, grooming facilities, health care facilities, institutions, laboratories, medical offices, nursing homes, prisons, schools, small animal facilities, veterinary clinics, Restoration, decontamination, and remediation sites (fire, water, sewage, trauma scene).

Veterinary Practice / Animal Care / Animal Laboratory / Zoos / Pet Shop / Kennels Disinfection Directions: For cleaning and disinfecting hard nonporous surfaces: equipment used for animal food or water, utensils, instruments, cages, kennels, stables, catteries, etc. Remove all animals and feeds from premises, animal transportation vehicles, crates, etc. Remove all litter, droppings and manure from floors, walls and surfaces of facilities occupied or traversed by animals. Empty all feeding and watering appliances. Thoroughly clean all surfaces with soap or detergent and rinse with water. Saturate surfaces with Contec Citric Acid Disinfectant for a period of 10 minutes. Wipe or allow to air dry. Thoroughly scrub all treated surfaces that come into contact with food, including equipment used for feeding or watering, with soap or detergent and rinse with potable water before reuse. Do not house animals or employ equipment until treatment has been absorbed, set or dried.

ALLERGENS: Wipes away dirt, germs and allergens†. Allergen†. This product removes the following allergens: cat dander, cockroach debris / matter / particles, dog dander, dust mite debris / matter / particles, pet dander, pollen particles from Timothy grass, grass.

TYPES OF SURFACES / USE SITES: Ambulance patient care surfaces, Appliances, Baby/infant care surfaces, Backboards, Bathing units, Bathrooms, Bed railings, Blood pressure devices, Cabinets, Chairs, Changing tables, Children's toys, Chrome, Computers, Counters, CPR manikins, CT equipment/surfaces, CuVerro™ antimicrobial copper, Doorknobs, Exam tables, Exercise equipment, Floors, Furniture, Garbage cans, Glass, Glazed ceramic tile, Hand railings, Highchairs, Infant care equipment, Isolettes, Keyboards, Kitchen surfaces, Laboratory equipment / surfaces, Lights, Medical equipment surfaces / devices, MRI equipment surfaces, Neonatal / nursery surfaces, non-porous athletic mats, Obstetrical equipment surfaces, Operating room lights / tables, Patient transport surfaces, Pet habitats, Physical therapy equipment, Plastic, Play-care equipment, Refrigerators, Respirators, Respiratory care equipment surfaces, Sealed granite, Shower floors / walls, Sinks, Soaking containers, Stainless steel, Stalls, Surgical equipment surfaces, Tables, Telephones, Toilets, Training tables, Transportation equipment surfaces, Trays, Walls, Work stations, Wrestling mats, X-ray equipment surfaces.

TO CLEAN, DEODORIZE AND REMOVE DEBRIS FROM CARPETS AND FABRICS: To Spot Clean: Blot or scrub area to be treated with Contec Citric Acid Disinfectant to remove soiling. Wipe dry with clean cloth or towel. **To Deep Clean Carpets, Upholstery and Fabrics:** Vacuum to remove loose dirt and soiling. Thoroughly moisten with product using coarse tip pressure sprayer and rake or brush into nap. Clean treated areas using clean water extraction method, such as portable or truck-mounted carpet cleaning machine or upholstery cleaning machine. Allow to dry thoroughly prior to using. Always spot test in inconspicuous area prior to application for compatibility.

DEODORIZING AND CLEANING OF CARPET/FABRICS: To Deodorize and Clean Carpets, Fabrics, and other Materials: Contec Citric Acid Disinfectant can be used on most fabrics that can be cleaned with water-based cleaners. Not for use on wool, silk, or suede. Always test for color fastness and material compatibility in an inconspicuous area prior to use. Apply by sprayer, to ensure even coverage and allow for 10 minute contact time. Do not rinse, use hot water extraction wand to remove product from treated surfaces and then dry stroke carpet to remove excess moisture. Carpet can be air dried or fans may be used if carpet needs to dry more quickly.

Storage and Disposal: Store airtight at room temperature. Nonrefillable container. Do not reuse or refill this empty container. Offer empty container for recycling. If recycling is not available, discard in trash.

CONTEC® CITRIC ACID DISINFECTANT

BROAD SPECTRUM HOSPITAL DISINFECTANT

EPA Reg. No. 34810-35-71670
 EPA Est. No. 34810-MO-001

ACTIVE INGREDIENTS

Citric Acid 0.60%

OTHER INGREDIENTS 99.40%

TOTAL 100.00%

For a full list of ingredients, visit
www.contecinc.com/saferchoice/citricaciddisinfectant

- Botanical
 - Dye-free
 - Fragrance free
 - Bleach-free
 - Non-irritating
 - Naturally derived active ingredient
- Tuberculocidal
 - Germicidal
 - Virucidal*
 - Fungicidal
 - Ready-to-use
 - Carpet Deodorizer

Keep Out of Reach of Children

1 U.S. Gallon (3.785 L)

Part No. CAD1284

